IB 4G M-L cuff s V32 3924

Revision Date: 2024-08-26

Microlife Soft Cuff M-L

Catalogue Number: CA3-W0-975

**Description**

- ① Cuff
② Cuff tube
③ Cuff Connector A
④ Cuff Connector B

Choosing the connector

- For blood pressure monitors that have a white, silver or dark coloured cuff socket on the device, use **Connector A** (3) with the grey connector pin.
 - For blood pressure monitors that have a yellow coloured cuff socket on the device, use **Connector B** (4) with the yellow connector pin.
- To change the connector, pull the connector out of the cuff tube and insert the required connector into the cuff tube as far as it will go.

Cleaning the cuff

- Remove the cuff connector (3) from the cuff tube (2) and carefully pull the bladder through the opening at the edge of the cuff cover.
- Hand wash the cuff cover in soapsuds; not hotter than 30 °C.
- Completely dry the cuff cover by linen drying.
- Loop the cuff tube back through its opening and carefully place the bladder flat in the cuff cover.
- Reattach the cuff connector on the cuff tube.

The bladder must lay straight in the cuff cover, not folded.

Do not use fabric softener.

WARNING: Do not wash the cuff in a washing machine or dishwasher!

WARNING: Do not dry the cuff cover in a tumble dryer!

WARNING: Under no circumstances may you wash the inner bladder!

The cuffs are sensitive and must be handled with care.

Guarante

- The cuff has a functional guarantee (bladder tightness) for 2 years.
- Not included are sleeve and cuff enclosure due to wear.
 - The guarantee is only valid on presentation of the purchase receipt.
 - The guarantee does not cover damage caused by improper handling, accidents or non-compliance with the operating instructions.
- The cuffs have an average service life of 2 years or 5000 measurements, whichever comes first.

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